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ENG – Chlamydia and gonorrhoea test

- Perform the test at least 5 days after having unprotected sex.
 - Do not urinate for 2 hours before taking your urine sample.
 - **Do not wash your genitals before taking the sample.**
1. Take the disposable plastic cup found in the box and press in the sides to return it into its original shape. Catch into the cup the first urine that comes out when you start to urinate. Half a cup is more than enough.
 2. There is a screw-capped sample tube in the package. Make sure that tubes preservative solution lies on the bottom of the tube. Preservative solution shouldn't be discarded.
 3. Open the tube and take it disposable pipette out of its plastic packaging. Draw some urine into the pipette by pressing in its thick end and then releasing the pressure when the tip of the pipette is immersed in the urine in the cup.
 4. Use the pipette to transfer enough urine into the tube so that the surface rises to between the black bars. Pour the rest of the urine into the toilet.
 5. To prevent the urine from leaking out during transportation, screw in the cap of the sample tube firmly. Turn the tube up and down 3–4 times to mix the urine with the preservative.
 6. Put the tube into the resealable plastic bag. Do not remove the small piece of filter paper inside the bag. Seal the bag and put it into the box.
 7. Close the box carefully by inserting the wing-lets into the slots in the corners. Send the box in via regular mail. The postage is prepaid and no stamp is needed.

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CHECKING THE ANALYSIS RESULT

Once your sample has been analysed, you'll receive a text message. You can check your results by logging in to the page [See test results](#) with your bank ID.

Please note that if you ordered the test from a website for your municipality or FSHS, you will need to log in from that specific page to see your results.